

ACTIVATING BLUETOOTH MODE

1. Press and hold soft key #1 (blue button in upper left corner) to enter settings menu.
2. Using joystick, scroll down to "Bluetooth", right command of joystick to select.
3. Scroll to desired Bluetooth device ("R-net Mouse 1" or "R-net Mouse 2"), right or left command of joystick to turn <on>.
4. Scroll to "Exit" at bottom of screen, right command to "Exit" to settings menu.
5. Scroll to "Exit" at bottom of settings menu, right command to "Exit".
6. Power chair off/on twice.

PAIRING PERMOBIL JOYSTICK MODULE WITH DEVICE

1. Press and hold soft key #3 (blue button in upper right corner) to enter Bluetooth Mode ("M3"). Option to use power/mode toggle to access Mode 3.
2. Select desired device, if more than one device is active, by scrolling and selecting with right command of joystick.
3. Put in discoverable mode by holding joystick forward for 15 seconds or until you hear a beep, repeat by holding joystick in reverse for 15 seconds or until you hear a second beep.
4. Enter Bluetooth settings on device being paired, and if required search for new device. Pair with "R-net Mouse 1" or "R-net Mouse 2". If prompted for a passcode, enter 0000.

DEFAULT SETTINGS IN "MOUSE MODE"

- 2 Softkey #2 – Short hold is left click; long hold is double left click
- 3 Softkey #3 – Short hold is return to drive mode ("M1")
- 4 Softkey #4 - Short hold is right click; long hold takes you to home screen on an android device
- 5 Speed toggle second function – Scroll up/down



IF USING WITH OMNI DISPLAY

- Follow instructions as outlined above for activating Bluetooth mode and pairing with device.
- "Bluetooth" should show in Omni user menu. User will select "Bluetooth" from user menu through right command of alternative drive control.
- If user only has one device active, that device will be selected and ready for use.
- If more than one device is active, reverse command will scroll down the device list and right command will select device. If user cannot see the Permobil Joystick Module (PJSM), he/she will need to memorize order of devices on the PJSM, and count beeps to know what device is active.

PC PROGRAMMING OPTIONS

- Enable "Nudge" to allow quick hits of input device for mouse clicks
- Program mono jack mode port to use external switches for mouse clicks
- Customize screen graphics
- Personalize names of Bluetooth devices